



Quick Start Guide

PREDICTABLE®
perfection™
3Dose™


1 Locking handle in FREE-mode

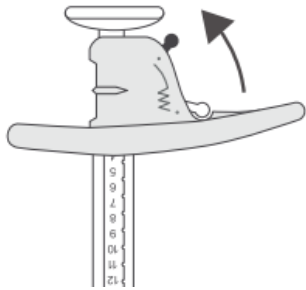


2 Pull-up dilution
Remove air-bubbles

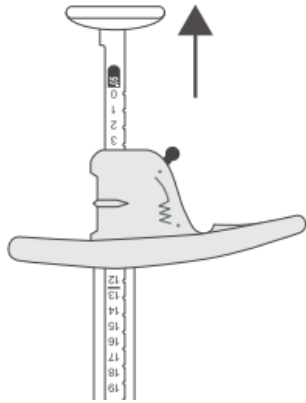


3 Rotate plunger to set scale
See dilution table on the box

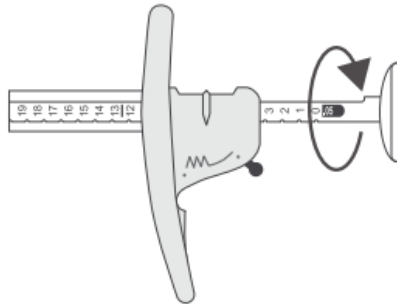

4 Locking handle in OPERATION-mode

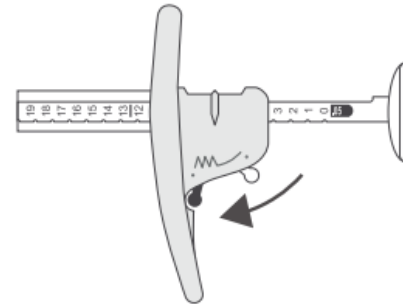


5 Ready to inject

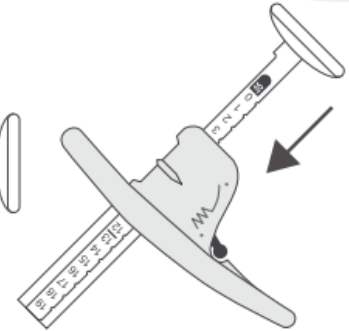

Instructions

1. Move the locking handle up to freely move the plunger
2. To pull up any dilution without air bubbles:
 - a. start with plunger in front position
 - b. pull up a little and push back
 - c. pull up the volume you need
3. Place the injection needle
4. Select the scale
5. Move the locking handle down
6. Ready to inject!

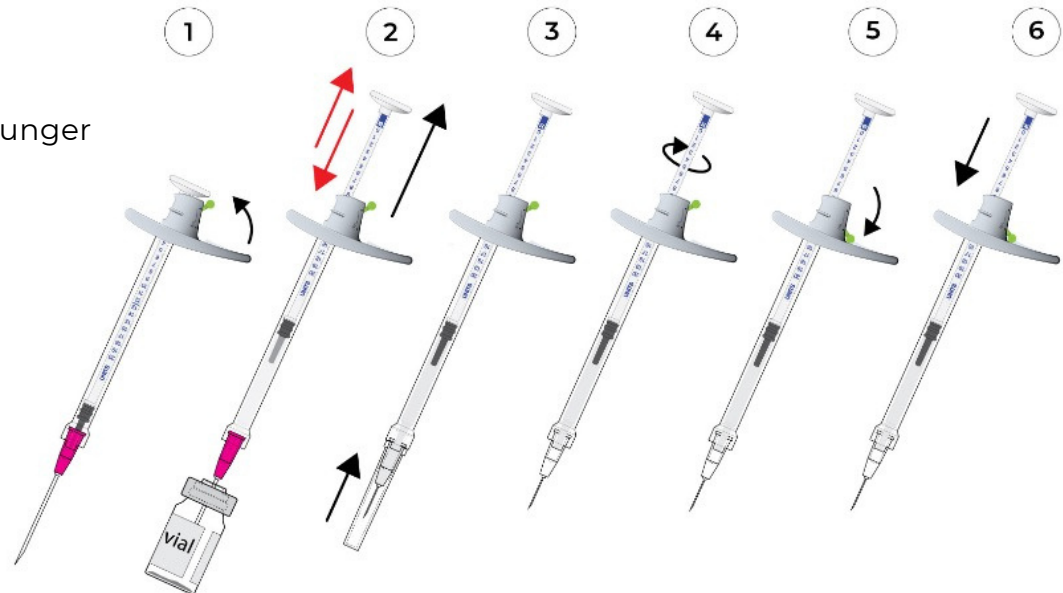

Brand / Manufacturer	Vial [units]	Solvent Added Sodium Chloride [ml]	Reconstitution Solution [units/0.1 ml]	3Dose™ Syringe - green - orange	3Dose Plunger Setting - green - black - blue	Number of units per click <i>X units = 1 click</i>
Botox® Allergan Letybo® Croma Xeomin® Merz Jeuveau® Evolus	50 units	(2.5:1)	1.25 ml	Green 3DOSE125	green: 0.0125 black: 0.025 blue: 0.05	0.5 unit
	100 units		2.50 ml			1 unit
Botox® Allergan Xeomin® Merz	50 units	(2:1)	1.0 ml	Orange 3DOSE100	green: 0.01 black: 0.02 blue: 0.04	0.5 unit
	100 units		2.0 ml			1 unit
	50 units	(1:1)	0.5 ml	Orange 3DOSE100	green: 0.01 black: 0.02 blue: 0.04	1 unit
	100 units		1.0 ml			2 units
Daxxify® Revance Tx	50 units	(1.25:1)	0.6 ml	Green 3DOSE125	green: 0.0125 black: 0.025 blue: 0.05	1 unit
	100 units		1.25 ml			2 units
Dysport® Galderma	300 units	0.6 ml	50 units / 0.1 ml	Orange 3DOSE100	green: 0.01 black: 0.02 blue: 0.04	5 units
	300 units	1.5 ml	20 units / 0.1 ml	Green 3DOSE125	green: 0.0125 black: 0.025 blue: 0.05	2.5 units
	500 units	2.5 ml	20 units / 0.1 ml	Green 3DOSE125	green: 0.0125 black: 0.025 blue: 0.05	5 units
	300 units	2.0 ml	15 units / 0.1 ml	Orange 3DOSE100	green: 0.01 black: 0.02 blue: 0.04	1.5 units
	300 units	3.0 ml	10 units / 0.1 ml	Green 3DOSE125	green: 0.0125 black: 0.025 blue: 0.05	3 units
	300 units	3.0 ml	10 units / 0.1 ml	Green 3DOSE125	green: 0.0125 black: 0.025 blue: 0.05	6 units